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INSTRUCTIONS FOR USE

BAUSCH + LOMB Artelac Advanced

With Artelac Advanced, you have chosen a product that will moisturize, refresh and revitalize your eyes. The surface of your eyes will regain its moisture, giving you long-lasting relief from the symptoms of dry eyes. Artelac Advanced contains hyaluronate, a natural polymer that also exists in structures of the human eye and the natural tear film. Artelac Advanced is therefore well tolerated. Artelac Advanced is suitable for moistening both soft and hard contact lenses and clearly improves the wearing comfort of your contact lenses the whole day long.

Please read these instructions for use carefully. They contain important information about the use of this product. Should you have any questions, please consult your chemist, your doctor or Laboratoire Chauvin - Bausch & Lomb.

What packaging units are available?

Artelac Advanced is available in packages of 30 single-dose vials of 0.5ml solution. 1ml solution contains 2 mg of hyaluronic acid and is free of preservative.

When should I use Artelac Advanced?

Due to various factors, the surface of your eyes often becomes dry. Symptoms such as a feeling of dryness, smarting or tiredness of the eyes often occur after working in front of computer screens or watching TV for a long time, or as a result of dry room air, heated air, air conditioning, cigarette smoke or draughts. Diseases, medications or treatments performed on the eyes can also play a major role in the development of dry eyes. Artelac Advanced promotes the humidification of the eye surface and serves as a moistening agent during the wearing of soft and hard contact lenses. It protects, refreshes and revitalizes your eyes.

How do I use Artelac Advanced?

Using Artelac Advanced is simplicity itself. Enclosed you'll find instructions on using the single-dose vials. Be careful to hold the single-dose vial as vertically as possible when dripping it into your eyes and do not touch the tip of the vial.

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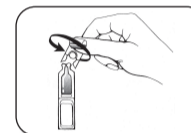
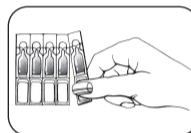
277

Avoid contact between the single-dose vial and your eye or contact lens to prevent possible injury or damage.

What do I need to remember when using Artelac Advanced?

To improve its tolerance, Artelac Advanced contains no preservative. Any remaining vial contents must be discarded after use. In very rare cases, a hypersensitivity reaction may occur. In this case, discontinue your use of the product. Do not continue using Artelac Advanced after the indicated expiry date and never use a single-dose vial that has been damaged.

What role does tear fluid play?

Your natural tear fluid is spread over the surface of your eye each time you blink, to protect the eye from drying out, aid the healing of superficial wounds to the cornea and prevent corneal damage.

A disturbance of tear production gives rise to various symptoms such as a sensation of dryness, smarting, the feeling that there's something in your eye, irritated and red eyes, sticky eyelids in the morning and in many cases watering eyes.

These symptoms have a variety of causes: Prolonged computer work, dry heated air, smoky or dusty air at work or contact lens wear. Certain illnesses, as well as certain medications, or the preservatives contained in many eyedrops can also cause a feeling of dryness.

How can Artelac Advanced help my eyes?

Artelac Advanced, containing hyaluronic acid, a natural substance in the body, improves the stability of the tear film. Your eyes are supplied with soothing humidity for longer.

Artelac Advanced consists of transparent drops, which permit clear sight without streaks. Artelac Advanced comes in practical, easy-to-use and hygienic single-dose vials. Preservative-free, easy to insert and in the ideal dose size, you can take them with you wherever you go.

What can I do to help myself?

- Take regular short breaks from computer work.
- Remember to blink frequently.
- Get regular fresh air.
- Drink plenty of fluids.
- Do not point air blowers (e.g. in your car) directly into your eyes.

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© Bausch & Lomb Incorporated.
®/™ are trademarks of Bausch & Lomb
Dr. Gerhard Mann - Chem.-Pharm. Fabrik GmbH