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Please read these instructions for use carefully – they contain important information on how to use this product. If you have any questions, please ask your pharmacist, doctor, eye care professional or contact us.

BAUSCH+LOMB

Artelac® Complete MDO®

Composition: 0.24% sodium hyaluronate, carbomer, glycerol, lipid component (medium chain triglycerides), sodium hydroxide and water for injection

Intended use: Eye lubricant

What is Artelac® Complete MDO?

Artelac® Complete MDO is an opaque eye drop solution that contains 0.24% hyaluronic acid (as sodium hyaluronate), carbomer, glycerol a lipid component (medium chain triglycerides), sodium hydroxide and water for injection.

The advanced formulation possesses a very high tolerability as it is free of preservatives and free of buffers.

The eye drops are presented in a preservative free multi-dose system (PFMD), which is easier to use and safe. Once opened, it can be used for 6 months.

What is the application of Artelac® Complete MDO?

Artelac® Complete MDO is an eye lubricant that supports all three layers of the tear film to improve the moistening of the ocular surface in the presence of dry eye sensation.

When you should use Artelac® Complete MDO?

Artelac® Complete MDO is – inspired by the design of the natural tear film. It contains the natural moisturizer hyaluronic acid (as sodium hyaluronate), which is found in the structures of a healthy human eye and the natural tear film and two additional moisturizers: Glycerol and Carbomer. Containing an additional lipid-component (medium chain triglycerides) **Artelac® Complete MDO** is supporting all three layers of the tear film: lipid, aqueous and mucin layer.

Artelac® Complete MDO is designed to remain on the surface of the eye providing intensive moisturization for immediate, long-lasting symptom relief. It is an ideal product for everybody suffering from a range of symptoms of dry eyes such as tired, stressed, gritty, burning and/or tearing eyes. Dry eyes are caused by

- External factors, like extensive computer use, air conditioning, heating, cigarettes smoke, as well as windy outdoor environments and/or wearing contact lenses.
- Other factors, like certain drugs (such as the contraceptive pill, antihistamines, beta-blockers, antispasmodics and diuretics) or certain diseases that play a role in the development of ocular dryness.

How often and how long should you use Artelac® Complete MDO?


Instill one drop of **Artelac® Complete MDO** into the eye as the need arises. You can use it every day and as often as you want. There is no limit of duration of use.

How to use Artelac® Complete MDO?

The patented pump mechanism of **Artelac® Complete MDO** allows the drop solution to be preservative free, but it may not look like the drop bottles you have used in the past.


General note: Wash your hands before each use. Avoid contact of the tip of the bottle with your fingers or your eyes to prevent any possible contamination of the solution or injuries.

1. Before each use, remove the protective cap from the tip of the bottle with a rotating movement.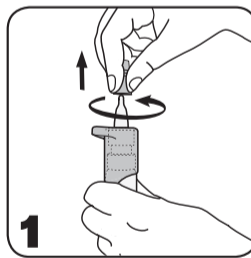


2. Attention! Before first use and if the bottle has not been used for a while:

Point the bottle with the tip downwards and press repeatedly the pump mechanism until a drop is released.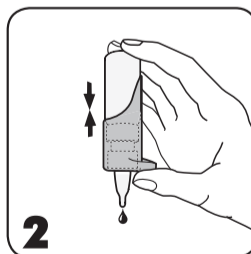


3. Hold the bottle as indicated:

Place your thumb on the ergonomic thumb support and enclose the lower part with your fingers. Two fingers should be placed on the bottom of the bottle.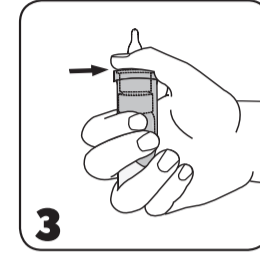

4. Point the bottle with the tip downwards. Lean back slightly your head. Gently pull your lower eye-lid down with a finger of your free hand, hold the bottle in a vertical position over your eyes as. Press gently the pump mechanism together to apply one drop into the eye.

Close your eye afterwards and blink in order to optimally spread the drop on the surface of your eye.

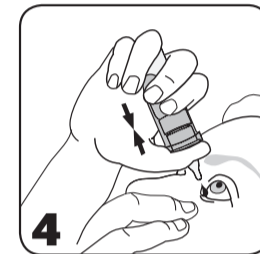5. Immediately after use, shake-off any possible residual drops and replace the protective cap on the tip of the bottle.

Due to the bottle design, a small amount of solution remains in the bottle. However, the specified filling volume can be completely removed from the bottle.

When you should not use Artelac® Complete MDO?

Do not use this product if you are hypersensitive to any of the ingredients.

Important information about using Artelac® Complete MDO:

- Local hypersensitivity reactions have been very rarely reported. Treatment should discontinue in such cases. If you experience persistent discomfort or irritation, stop using the product and consult your doctor.
- This product may cause temporary blurred vision after instillation which can have negative impact on driving vehicles or operating machines.
- Do not touch the eye surface with the tip of the bottle to prevent any possible injury of the eye surface or contamination of the solution.
- Shake-off any possible residual drops after application.
- Once opened, do not share the bottle with anyone else.
- Careless use of the product contrary to the instructions for use could result in slipping a product from hand and may cause eye injury
- Remove contact lenses before usage.
- Wait approximately 15 minutes before using another eye product or before replacing contact lenses.
- Discard the medical device after 6 months once opened.
- Do not use after the use-by date.
- Do not use if the package or bottle is damaged.
- **Artelac® Complete MDO** is filled in a bottle with small plastic parts that could be swallowed by small children. Danger of suffocation. Therefore, **Artelac® Complete MDO** should be kept out of reach of children.

How to store Artelac® Complete MDO?

Store at a temperature below 25 °C.

The design of the bottle allows you to use **Artelac® Complete MDO** for 6 months after opening.

Presentation:

Artelac® Complete MDO is available in packs of 10 ml or 15 ml eye drops. **Artelac® Complete MDO** is available in convenient, preservative free single dose units (SDU) as well.

Symbols

Legal Manufacturer:
Dr. Gerhard Mann
Chem.-pharm. Fabrik
GmbH
Brunsbütteler Damm
165-173
13581 Berlin
Germany

Symbol

Temperature limit

Sterilized using aseptic processing techniques

Do not use if package is damaged

Can be used for 6 months after first opening

Title

Consult instructions for use

Used-by date

Batch code

Caution: Consult the "Instructions for Use" for important cautionary information such as warnings and precautions

Manufacturer

Artelac® Complete MDO, Text Version: 999 EU-03, Dated: August 2018

®/™ are trademarks of Bausch & Lomb Incorporated or its affiliates

© Bausch & Lomb Incorporated
P1MEAST01 43071PB999/1-ME